

## OSTEOTRON IV QUICK SET-UP GUIDE

*(This guide is intended as a quick reference once the user manual has been read in full)*

1. Your device will come with rechargeable batteries and a charger. Place the batteries into the back of the device.
2. If using only one probe, ensure it is channel 1.
3. Apply the ultrasound gel to the probe.
4. Position the probe on the area to be treated according to physician's instruction and secure with strap or wheat bag.
5. Press and hold the power switch (for approx 1.5 sec)
6. The device sounds a series of beeps and then the treatment begins with the timer counting-down.
7. If the probe does not have full contact with the skin the treatment won't begin. Wiggle the probe to get better contact, apply more gel if required, and turn on and off again.
8. Standard settings are 30mW/cm<sup>2</sup> for 20 minutes. If the device displays something different, please contact Medi-Rent or your surgeon.  
Your screen will look like this if one probe is used:
9. Your screen will look like this if two probes are in use:
10. To pause, press the stop button
11. To resume, press the stop button again
12. When the treatment time reaches 0 the buzzer will sound and treatment will cease.
13. Make sure the device is turned off, remove the probe from the treatment area and wipe the gel from the probe. Recharge batteries when required.



Two probe leads inserted

